

Fold-a-Way Spring™

Play Tray Fitting Instructions

The Fold-a-Way play tray comes with two fixing brackets which attach to the frame of the Fold-a-Way:

1 Left hand bracket

1 Right hand bracket

Using the illustration as a guide fit the left hand bracket to the left hand side of the Fold-a-Way (viewed from the front).

Remove the screws (A) from the bracket and position over Fold-a-Way frame at the desired height..


Replace screws and tighten until the bracket is held firmly onto the frame.

Repeat procedure for the right hand bracket. Making sure both brackets are level.


Connect play tray to the brackets and lock at the desired height using screw (B).

Using locking nut (C) you can alter the angle of the play tray to keep it horizontal no matter how reclined the Fold-a-Way is.

Left hand bracket

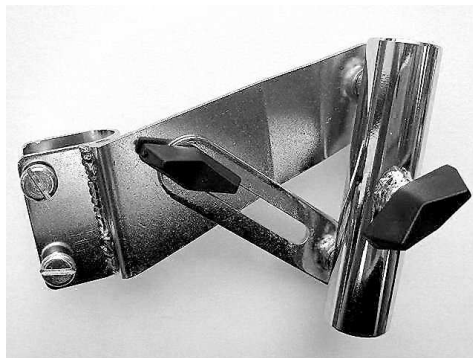

Right hand bracket

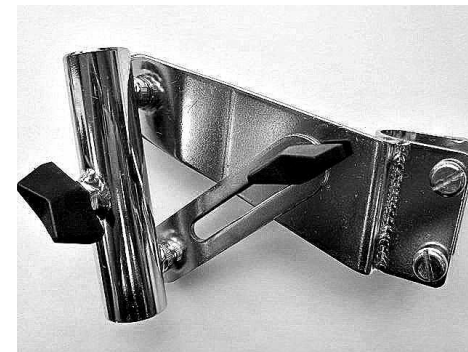

Adjust distance of play tray from Snugseat using (D) and tighten to lock



(D)

Adjust height of play tray using (B) and tighten to lock

(B)

Adjust angle of play tray using (C) and tighten to lock

(C)

(A)



Tendercare Ltd
PO Box 3091
Littlehampton
BN16 2WF
www.tendercareltd.com
Info@tendercareltd.com