

Mini Tilt

Play Tray Fitting Instructions

The Mini Tilt play tray comes with two fixing brackets which attach to the Mini Tilt armrests:

1 Left hand angle adjustable bracket

1 Right hand angle adjustable bracket

Please ensure the armrests are in the correct position on the wheelchair.

Using the illustration as a guide fit the left hand bracket to the left hand armrest of the Mini Tilt (viewed from the front).

Remove the black end cap on the front of the armrest

Remove the front screw (A) from underneath the armrest pad

Slide the fixing bracket over the armrest tube, and ensure the black thumb screw (B) is on the outer side of the chair.


Replace the front armpad screw (A), ensuring all holes are aligned.

Position tray and adjust accordingly.


Tendercare

Tendercare Ltd
PO Box 3091
Littlehampton
BN16 2WF
www.tendercareltd.com
Info@tendercareltd.com


Left hand bracket

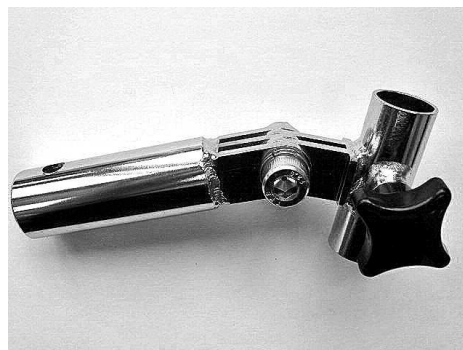

Right hand bracket

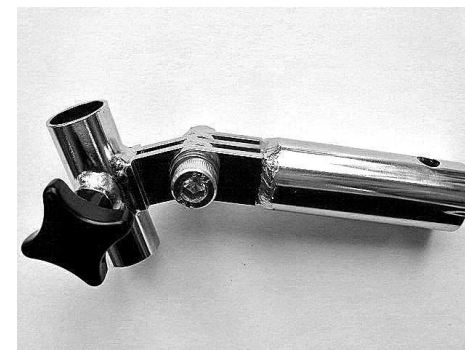


Play tray

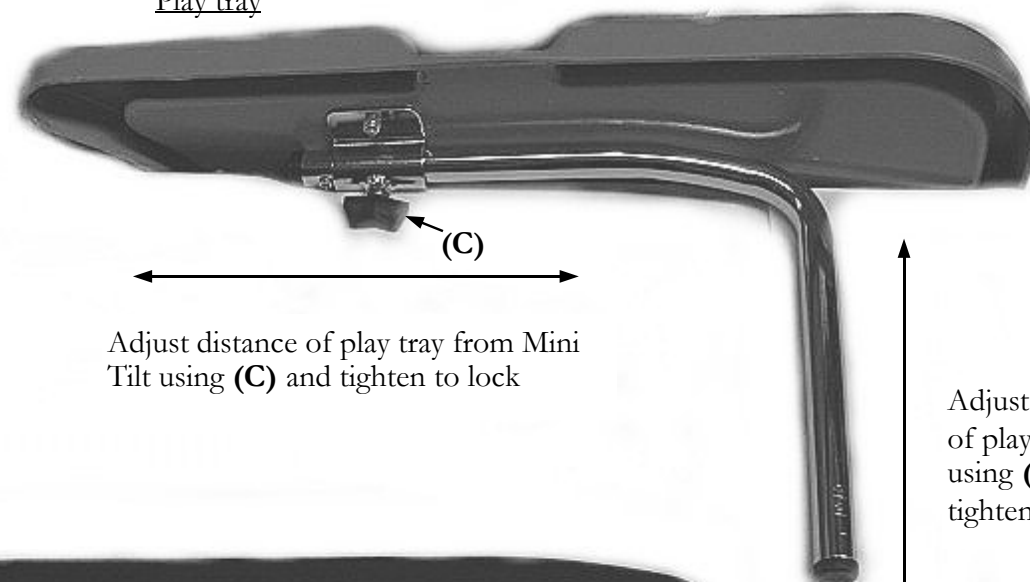

Adjust distance of play tray from Mini Tilt using (C) and tighten to lock

Adjust height of play tray using (B) and tighten to lock

Armrest

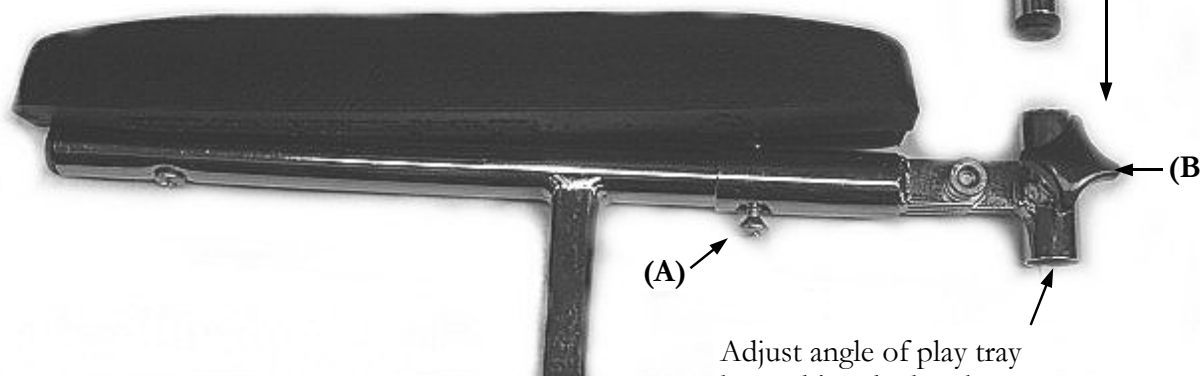

(A)

(B)

Adjust angle of play tray by pushing the bracket up or down

Note. All trays are despatched to fit onto wheelchairs with outriggered armrests. If your wheelchair is fitted with straight armrests, remove the tray legs and fit onto opposite sides.