

# Special Seating Seat Frame

## Play Tray Fitting Instructions

The Modular play tray comes with two fixing brackets which attach to the frame of the Special Seating Seat Frame:

- 1 Left hand bracket
- 1 Right hand bracket

Using the illustration as a guide fit the left hand bracket to the left hand side of the seat frame (viewed from the front).

Remove the screws (A) from the bracket and position over Seat Frame at the desired height.


Replace screws and tighten until the bracket is held firmly onto the frame.

Repeat procedure for the right hand bracket. Making sure both brackets are level.


Connect play tray to the brackets and lock at the desired height using screw (B).

Using locking nut (C) you can alter the angle of the play tray to keep it horizontal.


Left hand bracket

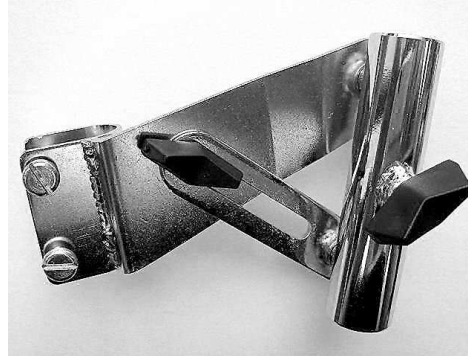

Right hand bracket

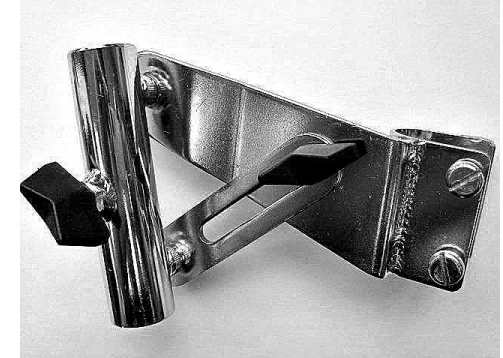

Adjust distance of play tray from Seat Frame using (D) and tighten to lock

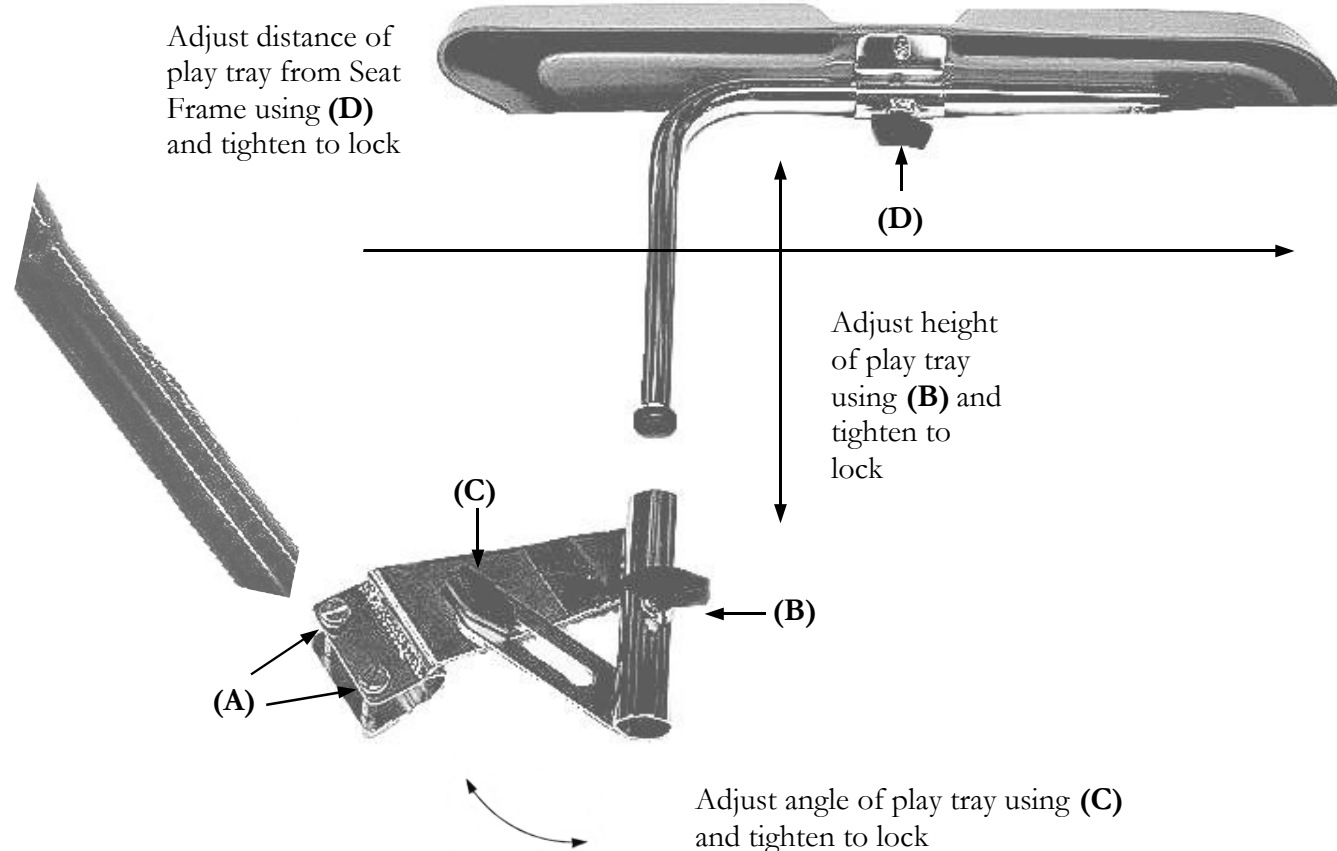

Tendercare Ltd  
PO Box 3091  
Littlehampton  
BN16 2WF  
www.tendercareltd.com  
Info@tendercareltd.com