

# Supplementary Interface Instructions


## Snappi Buggy

### Important

- This Supplementary Interface Instruction sheet **MUST** be read IN ADDITION to 'Fitting and Operating Instructions – Passive Locking Interface for use with MiniCAPS or CAPS II Seating Systems'.

### Fitting Instructions

1. The angle brackets should be bolted to the board before fitting the interface. Button head bolts without washers should be used, and a suitable threadlocking compound applied to these bolts before fitting.
2. Offer the interface board up to the triangular plates on the Snappi buggy to ensure the frame fits between the vertical leg of the brackets fitted to the bottom of this interface board.
3. Ensure the brackets are in line by using a straight edge (see below).
4. Remove the bolts that are positioned through the side triangular plates in the buggy and fit the interface. Refit the longer (35mm) bolts supplied, applying thread-locking compound before fitting.

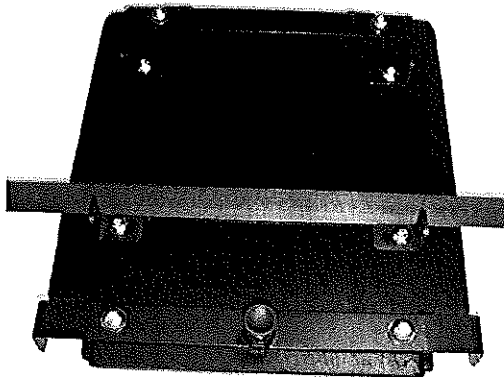

Active Design Ltd  
68k Wyrley Road, Birmingham. B6 7BN  
Tel: 0121 326 7506 Fax: 0121 327 8807  
E-mail: [technical@activedesign.co.uk](mailto:technical@activedesign.co.uk)  
Web Site: [www.activedesign.co.uk](http://www.activedesign.co.uk)

If you are unsure of any part of  
these instructions please contact  
us.