RECARO Sport Reha

Group I – III

- Ⓓ Montage- und Gebrauchsanleitung für Reha-Version
- Ⓖ Instructions for assembly and use of the Reha version
- ⒰ Instructions for assembly and use of the Reha version

1a

1b

2a

2b

3a

3b

4a

4b

5a

5b

6a

6b

7a

7b

8a

8b

9a

9b

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RECARO Sport Reha – Instructions for assembly and use of the Reha version

Dear RECARO Expert plus Reha user,

To make handling of your new children car seat safe, practical, and comfortable for both you and your child, please read these instructions and the instructions enclosed to the seat before use.

Should you have further questions or problems, please contact your specialist dealer or us directly.

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General instructions

Please note:

The 5-point safety belt integrated in the seat is intended for the use in ECE group I only. This requires that the seat must be secured with the vehicle's 3-point belt before.

Suitable for a body weight of up to 36 kg / 79.4 lb:

RECARO Sport Reha has successfully passed an overload test at Dekra (Dresden) on the basis of the requirements of ECE R 44/04. Thus, the RECARO Sport Reha is approved for the use as "restraint system for special needs ("S")".

Purpose / indication

The children car seat RECARO Sport Reha is designed for the transport of children with special needs in vehicles. The clinical pictures are amongst others:

- cerebral palsy
- hypotony syndrome of unspecific genesis
- muscular dystrophy
- different syndrome diseases

The car seat is characterized by its shell design and pad system, which provide for additional seating safety, stabilization of head and pelvis, and support / control of the feet.

Thus, the RECARO Sport Reha meets the needs of disabled children very well.

Application risks / contraindications

No application risks / contraindications are known.

5-point safety belt (group I)

Please note:

The 5-point safety belt is intended for the use in ECE group I only. This requires that the children car seat is secured with the vehicle's 3-point belt before.

The 5-point safety belt is already fitted in the seat. For the rehab version it is equipped with an additional crotch pad. For more information on the use of belts please consult the RECARO user manual provided with the seat.

Adjusting the belt length:

Slightly lift the lateral seat cover and release the fabric flap by opening the Velcro. Now you can see the central belt adjuster in the seat shell. To change the belt length, press the lever down and adjust the belt by either pulling the strap coming out of the front of the seat (shortening) or by pulling out both shoulder belts (lengthening).

Belt buckle:

Press the red dent of the buckle to open. Separate the two upper belt fasteners. To close the belt, bring the two upper belt fasteners together, insert them into the buckle from above and push together until they lock in place with an audible „Click“.

The cover for the crotch pad (zipper) can be removed and washed in the machine at up to 30° C / 86° F. Please undo the zip on the bottom crotch pad as well to remove the cover.

Fig. 1: Adjusting the belt length

Fig. 2: Opening the belt buckle („Click“)

Headrest

Pull the covers as tight as possible over the side parts of the headrest and attach them to the screws at the top.

Fig. 1: Pulling headrest over side part

Fig. 2: Attachment to screw

Lateral trunk support

Attach the lateral trunk support by placing it on the seat. Guide both Velcro strips behind the back cover and Velcro-fasten them behind the cover.

Just as required, the lateral trunk supports can be used with either the small or the large side facing upwards.

The cover can be removed and washed in the machine at up to 30° C / 86° F.

Fig. 1: Positioning on the seat

Fig. 2: Velcro fastening

Lateral seat support

Remove the crotch pad from the belt buckle. Attach the lateral seat supports by placing them on the seat and guiding the belt buckle through the slot provided in the support. Attach the supports to the rear of the seat surface with both press buttons. Re-attach the crotch pad to the belt buckle.

The cover can be removed and washed in the machine at up to 30° C / 86° F.

Fig. 1: Positioning on the seat / guiding belt buckle through slot **Fig. 2:** Attachment to press buttons

Footrest adapter (accessory)

item code: 866

Before assembly, clean the plastic surface on the underside of the seat with commercially available cleaning agents (e.g. water and rinsing agent) until it is completely free from grease.

Now remove the Velcro strips (fleece) from the adapter plate and attach them to the underside of the seat. The Velcro strips (hooks) remain on the adapter plate.

Position the seat flush in a 90° angle at the front of the adapter (handle of the adapter facing forwards) and tilt the seat over to velcro it to the adapter.

Footrest: The footrest is attached to the adapter.

Fig. 1: Attaching Velcro straps to the seat **Fig. 2:** Attaching seat to adapter

Footrest, short 12 cm / 4.7" (accessory)

item code: 806

Press the catch springs in at the rear ends of the sliding rods and push the tubes into the holes as far as they will go.

To stabilize the footrest on the vehicle floor adjust the height of the support bars by screwing them in or out **until they have contact with the vehicle**. Then lock the support bars with the hand knobs to prevent them unscrewing accidentally. Screw the hand knobs upwards as far as possible and tighten them fast.

Lower leg length:

The lower leg length is adjusted by changing the height of the footrest. Loosen both outer knurled nuts. Retighten them after each adjustment. To achieve the highest position possible, use the elevation block (+10 cm / 3.9") provided. It is attached centrally on the footrest by Velcro.

Foot angle:

Loosen the four screws left and right on the underside of the footrest to adjust the desired foot angle with the rails. Then retighten the screws.

Note:

Always secure the footrest against falling off with the hand knob provided with the user manual. The receiver for the hand knob (metal block with thread) is situated on the left tube of the footrest adapter.

Fig. 1: Inserting the support **Fig. 2:** Adjusting height to vehicle

Turning plate with footrest adapter (accessory)

item code: 812

Before assembly, clean the plastic surface on the underside of the seat with commercially available cleaning agents (e.g. water and rinsing agent) until it is completely free from grease.

Now remove the Velcro strips (fleece) from the turning plate and attach them to the underside of the seat. The Velcro strips (hooks) remain on the turning plate.

Position the seat flush in a 90° angle at the front of the turning plate (handle of the turning plate facing forwards) and tilt the seat over to velcro it to the turning plate.

The turning plate is pre-assembled for the attachment of the car seat on the right back seat of the vehicle (safe placing / removing of the child from the sideway in countries with right hand traffic). If the car seat shall be fixed to the left back seat the pivotal point of the turning plate has to be changed. This is done by simply dislocating the Allen screw from right to left.

Note:

The audible clicking into place of the turning plate just serves as orientation aid for the straight positioning of the seat on the turning plate.

Transport:

To transport the seat with turning plate, hold the handle of the turning plate with one hand and the back of the seat with the other. To reduce the load, seat and turning plate can also be transported separately.

Footrest:

The footrest is attached to the bottom part of the turning plate with the adapter.

Fig. 1: Attaching Velcro straps to the seat **Fig. 2:** Attaching seat to turning plate / dislocating the Allen screw

Seat wedge, bottom (accessory)

item code: 810

The seat wedge is positioned centrally below the car seat. The high edge is at the front to achieve an optimum rest position of the child.

The cover can be removed (zipper) and washed in the machine at up to 30° C / 86° F.

Please note:

Function of the turning plate is considerably reduced if being used in combination with the seat wedge. We therefore do not recommend using seat wedge and turning plate at the same time.

Fig. 1: Positioning seat on seat wedge **Fig. 2:** Final position

Table (accessory)

item code: 811

Mount the side parts of the table first, by pulling them over the round corners of the seat. The side with the larger fabric must face outwards. Attach the fabric to the wheels for the seat adjustment using the provided holes in the cover of the side parts. Then tuck the downwards overlaying fabric below the seat cover.

Now fasten the table to the Velcro straps of the side parts.

Note:

For reasons of space the crotch pad should be removed when using the table.

The cover can be removed and washed in the machine at up to 30° C / 86° F.

Fig. 1: Mounting the side parts **Fig. 2:** Attaching the table

Member of the International
Support Association for the
Rehabilitation of Children and Youths e.V.

Item code: 95570, status: 06/2009, subject to technical alterations

Walkmühlenstraße 1 · D - 27432 Bremervörde · Germany
Phone: +49 (0) 47 61 / 8 86-68 or -63 · Fax: +49 (0) 47 61 / 8 86-19
E-Mail: info@thomashilfen.de · www.thomashilfen.com